



Institutional Laundering Guidelines for the HALO® SleepSack® Swaddle and Wearable Blanket

The HALO® SleepSack® wearable blanket and SleepSack Swaddle are designed for In-Hospital use and can be laundered using the same methods as general institutional linens composed of similar fabrics and components.

HALO's In-Hospital products use high-quality polyester micro-fleece and cotton fabrics with heavy-duty, metal zippers, nylon snaps, or Velcro® fasteners.

Laundering Recommendations

Wash:

- Water temperatures up to 200 degrees Fahrenheit (93 degrees Celsius).
- To avoid graying of polyester micro-fleece fabric, use only peroxide ("color-safe") bleach. Chlorine bleach is acceptable for use with cotton fabrics.
- Avoid using a tunnel-press for water extraction on all zippered products to prevent zipper pull and slider damage.
- Fasten Velcro® closures before washing.

Dry:

- Dry at temperatures below 170 degrees Fahrenheit (77 degrees Celsius).